Fig. 1

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Zachte contactlenses

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Instructions for use

Soft contact lenses

Product name: see packaging label.
Material name: see packaging label.
Storage fluid: saline.
Intended use: the contact lenses are intended for daily wear.

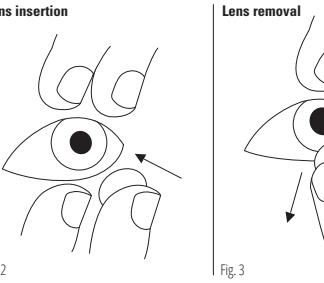
Fig. 2

Indications for use: correction of refractive ametropia (such as near and farsightedness) for users of 8 years and older.
Wearing schedule: the lenses are suitable for daily wear. Do not sleep in your lenses unless your eye care professional has advised it is safe to do so. Follow the wear and replacement schedule provided by your eye care professional.

Instructions for cleaning and use
Lens normal/inside-out (Fig.1)
Lens insertion (Fig.2)
Lens removal (Fig.3)

- The contact lenses and the contact lens case must be regularly and carefully rinsed, maintained, and replaced according to the instructions provided by your eye care professional. Do not use tap water to rinse the lenses. Saline (sterile salt solution) is preferred and allow your lens case to air dry while open.
- Use only the cleaning solutions recommended by your eye care professional to avoid unwanted interaction between the contact lenses and the cleaning solution
- Wash your hands thoroughly with soap and dry them carefully before handling lenses.
- Do not use in case the contact lens is damaged.

Warnings

Do not use and consult your eye care professional if any of the following conditions occur:

- Inflammation, infection, redness or irritation of the eye or eyelid.
- Changes in corneal sensitivity or reduced tear production.
- Any general disease that may affect the eye, or for which wearing contact lenses may increase the risk of complications.
- Intolerance or abnormal eye reaction to the wearing of contact lenses.

Report any changes in eye comfort while wearing the contact lenses to your eye care professional; these changes may include increased tearing, severe redness, poor vision or allergic reaction. Failing to seek timely medical treatment may result in permanent eye damage.

Do not use contact lenses if the contact lens wearer cannot or refuses to follow the instructions for safe contact lens use.

Do not expose your contact lenses to any water: tap, bottled, distilled, lake or ocean water. Never use non-sterile water (distilled water, tap water or any homemade saline solution). Tap and distilled water have been associated with Acanthamoeba keratitis, a corneal infection that is resistant to treatment and cure. Remove your contact lenses before swimming. There is a risk of eye infection from bacteria in swimming pool water, hot tubs, lakes and the ocean.

Precautions:

- Visit your eye care professional regularly to examine your eyes and contact lenses.
- In poor lighting conditions, bi- or multifocal lenses can create reflections that may impair vision. Ensure that your eyesight is unimpaired at dusk/night, particularly before operating a motorised vehicle.
- Users with a reading addition greater than +1,50 D should be carefully monitored by a eye care professional during the adaption phase. Visual acuity might be suboptimal during the adaptation phase.
- It is not safe to re-use soft contact lenses which the user has stored for a period longer than prescribed in the cleaning instructions. In this case, use new lenses and discard the old ones.
- This sterile contact lens has been sterilised by steam. Ensure that the packaging seal is not damaged or open when using lenses for the first time. Contact the supplier if this is not the case.
- Remove the lenses and rinse them thoroughly with the prescribed care system as soon as contact with cosmetics or cigarette smoke causes eye irritation
- Medications that are used over a long period of time may affect the wearability of the lenses. Be sure to inform your eye care professional of any medication you are taking.
- Pregnancy or the use of oral contraception drugs may reduce the wearing comfort of contact lenses. Consult your eye care professional if this occurs.
- Any serious health condition that has occurred in relation to the contact lenses should be reported immediately to the manufacturer and the competent authority of the European Member State in which the user is established
- Expiry date: See the package. Do not use after the expiry date.
- Recommended transportation conditions are between 5°C and 50°C and avoid direct sunlight. Recommended storage temperature are between 15°C and 35°C.
- Contact lenses can be disposed of via regular household waste unless local regulations require otherwise.

Explanation of possible symbols and abbreviations on the label:

-  Manufacturer.
-  Expiry date.
-  Lot number.
-  Steam sterilised.
-  Single sterile barrier system.
-  Medical Device.
-  Consult instructions for use.
- CE** The CE mark confirms that the product meets the standards laid down in the European legislation for Medical Devices.

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