Fig. 1

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Vormstabiele contactlens - orthokeratologie

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Instruction for use

Rigid gas permeable contact lens - orthokeratology

Name of the product: see packing label.
Name of the material: see packing label.
Storage solution: (if present) poloxamer 0,25%, EDTA 0,10% and polyhexanide 0,0002%, in a buffered isotonic and sterile solution.
Intended Use: the product is intended for overnight wear.
Indication for use: correction of myopia by temporary controlled reshaping of the cornea (orthokeratology).
Wearing schedule: the lenses are suitable for orthokeratology. Follow the wear and replacement schedule provided for your eye care professional.

Instructions for cleaning and use

Inzetten lens: (Fig. 1)
Removing the lens: (Fig. 2)
Centering the lens: (Fig. 3)

- The lenses and the contact lens holder should be regularly cleaned and looked after in accordance with the instructions provided by your eye care professional. Do not use tap water to rinse the lenses. Saline (sterile salt solution) is preferred and allow your lens case to air dry while open.
- Use only the care system recommended by your eye care professional to avoid unwanted interaction between the contact lenses and the care system. If lenses are not being used for a longer period:
 - Clean the contact lenses thoroughly.
 - Keep the lenses dry in a clean and absolutely dry contact lens holder.
 - Clean the lenses immediately before you use them again.
 - Leave the lenses for 8 hours in a storage solution for a better comfort.
 - Wash your hands thoroughly with water and soap and dry them carefully before handling lenses. Do not use in case the contact lens is damaged. Please dispose of contact lenses via normal household waste unless local regulations requires otherwise.

Warning

Do not use and consult your eye care professional if any of the following conditions occur:

- Inflammation, infection, redness or irritation of the eye or eyelid.
- Change in cornea sensitivity or reduced tear production.
- Any general health condition that has occurred in relation to the contact lenses may increase the risk of complications for replacement in case of your lenses get damaged or lost, and to guaranty you will always have a good vision in all situations.
- Intolerance or abnormal eye reaction to the wearing of contact lenses.

Report any changes in eye comfort while wearing the contact lenses to your eye care professional; these changes may include increased tearing, severe redness, poor vision or allergic reaction. Failing to seek timely medical treatment may result in permanent eye damage.

Do not use contact lenses if the contact lens wearer cannot or refuses to follow the instructions for safe contact lens use.

Parental guidance is required for minors and parents/guardians should be informed by the eye care professional concerning risks and benefits of wearing contact lenses.

Do not expose your contact lenses to any water; tap, bottled, distilled, lake or ocean water. Never use non-sterile water (distilled water, tap water or any homemade saline solution). Tap and distilled water have been associated with Acanthamoeba keratitis, a corneal infection that is resistant to treatment and cure. Remove your contact lenses before swimming. There is a risk of eye infection from bacteria in swimming pool water, hot tubs, lakes and the ocean.

Precautions:

- Visit your eye care professional regularly to examine your eyes and contact lenses.
- When removing the lens, the contact lens must move freely on the eye and swim in the tear film. Therefore, use a moisturising liquid as recommended by your eye care professional and blink a few times until the lens moves freely when removing lenses.
- When using for the first time, ensure that the packaging seal is not damaged or open. If it is not the case, contact the supplier.
- Remove the lenses and rinse them thoroughly with the prescribed care system as soon as contact with cosmetics or cigarette smoke causes eye irritation
- Pregnancy or the use of oral contraception drugs may reduce the wearing comfort of contact lenses. Consult your eye care professional if this occurs.
- Medications that are used over a long period of time may affect the wearability of the lenses. Inform your eye care professional of any medication you are taking.
- Take care to have a spare set of lenses for replacement in case of your lenses get damaged or lost, and to guaranty you will always have a good vision in all situations.
- In poor lighting conditions, bi- or multifocal lenses can create reflections that may impair vision. Ensure that your eyesight is unimpaired at dusk/night, particularly before operating a motorised vehicle.
- Users with a reading additioin greater than +1.50 D should be carefully monitored during the adaptation phase.
- Visual acuity might be suboptimal during the adaptation phase.
- Before initial use, ensure that the specific instructions on use of the Ortho-K lens have been explained by your contact lens practitioner.
- During the adaptation time, annoying reflections can occur, which can cause a risk especially when driving at night. Therefore, users of Ortho-K lenses need to be closely checked for at least one eye care professional.
- Lenses are also advised to be checked that they have undisturbed vision prior to operating in traffic.
- The best guarantee to achieve a good and stable visual acuity, is to wear the lenses every night.

- It can take some time before the best effect is achieved by Ortho-K lenses (up to 3 weeks), in particular for high minus powers.
- It is recommended to consult your eye care professional regarding insertion of the Ortho-K lens.
- Take great care of hygienic treatment of your lenses, to avoid complications
- Follow the evaluation schedule established by your eye care professional to ensure safe use of the Ortho-K lens.
- Replace the Ortho-K lens on a yearly basis. Not following these instructions can result in unsafe use or ineffective results of the product.
- Any general health condition that has occurred in relation to the contact lenses should be reported immediately to the manufacturer and the competent authority of the European Member State in which the user is established.
- Expiry date: See the package. Do not use after the expiry date.
- Recommended transportation conditions are between 5° and 50°C and avoid direct sunlight. Recommended storage temperature are between 15°C and 35 °C
- Contact lenses can be disposed of via regular household waste unless local regulations require otherwise.

Explanation of symbols and abbreviations on the label:

- Manufacturer.
- Expiry date.
- Lot number.
- Steam sterilised.
- Instructions for use.
- The CE mark confirms that the product meets the standards laid down in the European legislation for Medical Devices.

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